pfmmedical

Quality and Experience

Jet-Vac® 600D, 1000D, or 1500D

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www.pfmmedicalusa.com

U.S. Design Pat. No. D818119; Canadian Design Pat. No. 1722153

PFM Medical, the PFM logo, the color Green of the three-dimensional configuration of the accordion vacuum indicator and Jet-Vac are registered trademarks of pfm medical, inc.

© 2017 PFM all rights reserved.

pfmmedical

pfm medical, inc.
1916 Palomar Oaks Way, Suite 150
Carlsbad, CA 92008
Tel.: (760) 758-8749
Fax: (760) 758-1167
www.pfmmedicalusa.com
customerservice@pfmmedicalusa.com

LS-00121-01-AC / January 2019

This page is left intentionally blank.

This page is left intentionally blank.

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Instructions for Use

Jet-Vac 600D, Jet-Vac 1000D, or Jet-Vac 1500D (600 mL, 1000 mL, or 1500 mL Evacuated Drainage Bottle with Drain Line)

Product Description

The Jet-Vac 600D, Jet-Vac 1000D, or Jet Vac 1500D bottle is an evacuated drainage bottle with pre-attached drain line. The bottle comes with a drain line, a needle-free sampling port, and luer connector at the end of the drain line. The following instructions are for the Jet-Vac 600 mL, 1000 mL, or 1500 mL bottle with drain line.

Each Jet-Vac 600 mL, 1000mL, or 1500 mL vacuum bottle contains the following features:

- Clear drain line with pinch clamp
- Luer connector with protective cap
- Green vacuum indicator
- Green tube with vacuum release clamp
- Needle-free sampling port
- Bottle labeled in 10 mL increments

Indication for Use

The Jet-Vac evacuated drainage bottle with drain line is a vacuum, single patient, portable suction device used to aspirate, remove, or sample body fluids.

Contraindications

There are no known contraindications.

Warnings

- **Do not reuse. Intended for single patient use only.**
- **It is not uncommon for the patient to feel some pain or cough when draining fluid.**
- **Keep the needle-free sampling port and luer connector clean and sterile.**
- **This product and its packaging have been sterilized with ethylene oxide. Ethylene oxide is a chemical known to the state of California to cause cancer, birth defects, or reproductive harm.**

Precautions

- Federal Law (USA) restricts this device to sale by or on the order of a physician.
- This product contains dry natural rubber.
- The vacuum indicator should be down and level with the "max" line; this indicates that the inside of the bottle contains vacuum.

Application

Using the Jet-Vac evacuated drainage bottle with drain line

Follow these step-by-step directions.

1. Pick up the Jet-Vac 600, Jet-Vac 1000, or Jet-Vac 1500 evacuated drainage bottle with drain line. Look at the green vacuum indicator on the top of the bottle. This indicator tells you if there is a vacuum in the bottle. The indicator should be down, not up. If the indicator is up, discard the bottle and use another.
2. Remove the twist tie that holds the drain line together.
3. Squeeze the pinch clamp on the drain line completely closed
4. Remove the protective cap from the luer connector on the drain line and align it in the center of the opposing luer and twist together to ensure the connection is closed.

5. Slide the white clamp on the tubing sleeve to release the vacuum.
6. Slowly release the pinch clamp on the drain line to begin draining the fluid.
7. The fluid will flow into the evacuated drainage bottle.
8. After the drainage procedure, squeeze the pinch clamp on the drain line fully closed and disconnect the drain line.

Note: The luer locking, needle-free sampling port may be used for fluid sampling during or after drainage procedures. To sample fluid, swab the luer locking, needle-free sampling port with a disinfectant and twist a needle-free syringe into the sampling port, hold the bottle upside down and remove the fluid.

Disposal

Dispose the Jet-Vac 600 mL, 1,000 mL or 1,500 mL evacuated drainage bottle with drain line in accordance with the facility's policies and procedures.

STERILITY

This device has been sterilized, and **is for single use only and is not to be reused.** pfm medical will not be responsible for any products that are resterilized, nor accept for exchange or credit any product that has been used by the patient or purchaser.

WARRANTY

pfm medical warrants that this medical device is free from defects in both materials and workmanship. The above warranties are in lieu of all other warranties, either expressed or implied, including any warranty of merchantability or fitness for a particular purpose. Suitability for use of the medical device for any procedure shall be determined by the user. pfm medical shall not be liable for incidental or consequential damages of any kind.

For explanation of the symbols on the labeling and packaging, visit www.pfmmedicalusa.com

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