

pfmmedical

Quality and Experience

ASEPT® Safety Drain Line Adapter

Instructions for Use

www.pfmmedicalusa.com

Instructions for Use

Safety Drain Line Adapter

Product Description

The ASEPT Safety Drain Line Adapter allows the use of ASEPT bottles with other well-known indwelling catheters on the market.

Indication for Use

The ASEPT Safety Drain Line Adapter is intended to be used with an indwelling catheter and an ASEPT bottle. This product is not to be used with an ASEPT catheter. For instructions on how to use an ASEPT drainage bottle with an ASEPT catheter please refer to the product specific IFU.

Contraindications

There are no known contraindications.

Warnings

- **Do not reuse. Intended for single patient use only.**
- **This product and its packaging have been sterilized with ethylene oxide. Ethylene oxide is a chemical known to the state of California to cause cancer, birth defects, or reproductive harm.**

Precautions

- Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.
- The Safety Drain Line Adapter must be connected to the drain line prior to accessing the catheter.

Application

Follow these step by step instructions

1. Attach the Safety Drain Line Adapter to the drain line
 - a. After properly preparing the ASEPT drainage bottle or bag by completely closing the clamp on the drain line, remove the cap from the male luer connector on the drain line tubing. Do not discard.
 - b. Hold the female luer end of the Safety Drain Line Adapter in one hand and the male luer connector of the drain line tubing in the other.
 - c. Attach the male luer connector on the drain line tubing to the female luer connector on the Safety Drain Line Adapter by firmly pushing the ends together and turning clockwise to tighten. Do not over tighten.
2. Attach the Safety Drain Line Adapter to the Catheter
 - a. Hold the catheter valve collar and the white Safety Stylet tip of the Safety Drain Line Adapter firmly in each hand.
 - b. Align the white Safety Stylet Tip of the Safety Drain Line Adapter straight on with the opening of the catheter valve.
 - c. Introduce the Safety Stylet Tip of the Safety Drain Line Adapter into the valve collar.
 - d. Advance the Safety Stylet Tip firmly forward until the Silicone Guard is flush against the valve collar. Do not advance to Safety Stylet Tip any further.

This product is now safe and ready to use. Please continue with the standard bottle or bag instructions to complete the drainage for this patient.

3. If a second bottle is needed to complete the drainage
 - a. If a second bottle is needed to complete the drainage, open a second sterile Safety Drain Line Adapter and a second sterile bottle.
 - b. Do not connect to a Drain Line Adapter that has already been used. Repeat steps 1 through 4.

Removal

Follow these step-by-step directions.

1. Remove the Safety Drain Line Adapter from the Catheter
 - a. After drainage is complete and the clamp has been closed, firmly grasp the valve collar and the white safety stylet tip of the Safety Drain Line Adapter.
 - b. With slight effort, detach the hubs by pulling straight back. If necessary, slightly twist the Safety stylet tip while pulling back to assist in detaching the Safety Drain Line Adapter from the catheter.
2. Remove the Safety Drain Line Adapter from the drain line
 - a. After the Safety Drain Line Adapter has been detached from the Catheter, firmly grasp the locking collar from the drain line tubing in one hand and the female luer end from the Safety Drain Line Adapter in the other.
 - b. Disconnect the drain line from the Safety Drain Line Adapter by turning the locking collar counter-clockwise and pulling straight back.
 - c. Replace the cap on the male luer connector on the end of the drain line tubing.

Disposal

Dispose of the Safety Drain Line Adapter and the used ASEPT drainage bottle(s) or bag(s) as per your facility protocol.

STERILITY

This device has been sterilized, and **is for single use only and is not to be reused**. Pfm medical will not be responsible for any products that are resterilized, nor accept for exchange or credit any product that has been used by the patient or purchaser.

WARRANTY

pfm medical warrants that this medical device is free from defects in both materials and workmanship. The above warranties are in lieu of all other warranties, either expressed or implied, including any warranty of merchantability or fitness for a particular purpose. Suitability for use of the medical device for any procedure shall be determined by the user. pfm medical shall not be liable for incidental or consequential damages of any kind.

For explanation of the symbols on the labeling and packaging, visit www.pfmmedicalusa.com

pfmmedical

Distributed by:

Pfm Medical, Inc
1916 Palomar Oaks Way, Suite 150
Carlsbad, CA 92008

Tel.: +1 (760) 758-8749

Fax: +1 (760) 758-1167

www.pfmmedicalusa.com

customerservice@pfmmedicalusa.com



Manufactured by:

Medquest Medical
Inc 45 Midpark
Crescent London, ON
N6N 1A9 Canada

LS-00163-01-AA / January 2020