**pfmmedical**

Quality and Experience

---

**ASEPT® 600 mL or 1,000 mL**

**Drainage Kit**

---

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---

**[www.pfmmedicalusa.com](http://www.pfmmedicalusa.com)**

pfmmedical, the pfm logo, the color Green of the three-dimensional configuration of the accordion vacuum indicator and ASEPT are registered trademarks of PFM Medical, Inc  
*Tegaderm® is a registered trademark of 3M Company.*  
© 2024 PFM Medical, Inc.

---

**pfmmedical**

PFM Medical, Inc.  
1916 Palomar Oaks Way, Suite 150  
Carlsbad, CA 92008  
Tel.: (760) 758-8749  
Fax: (760) 758-1167  
[customerservice@pfmmedicalusa.com](mailto:customerservice@pfmmedicalusa.com)

LS-00191-01-AG / November 2024

This page is left intentionally blank.

This page is left intentionally blank.

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### ASEPT 600 mL or ASEPT 1,000 mL Drainage Kit

#### Product Description

The ASEPT Drainage Kit is a complete vacuum bottle kit equipped with a pre-attached drain line for pleural or peritoneal drainage and items to ensure proper sterile wound dressing procedure when draining fluid through an ASEPT catheter. The following instructions are for the ASEPT 600 mL Drainage Kit or the ASEPT 1,000 mL Drainage Kit when draining fluid through an ASEPT catheter.

**Each ASEPT 600 mL Drainage Kit and ASEPT 1,000 mL Drainage Kit contain the following items:**

**One ASEPT 600 mL Drainage Bottle or 1,000 mL Drainage Bottle with the following features:**

- Clear drain line with pinch clamp
- ASEPT connector with protective cap
- Green vacuum indicator plastic cap
- Green tube with vacuum release clamp
- Bottle labeled in 10 mL increments

**One ASEPT Procedure Pack containing:**

- 2 Pairs of Gloves
- 3 Alcohol Pads
- Foam Catheter Pad
- 6 Gauze Pads
- Tegaderm® Self-Adhesive Dressing
- Emergency Slide Clamp

#### Indications for Use

The ASEPT Drainage Kit is an accessory to be used to connect to the ASEPT Pleural or Peritoneal Drainage Catheter to drain fluid from the chest or abdomen. Each ASEPT Drainage Kit also contains all dressing change items needed for any ASEPT Drainage Catheter.

#### Indications for Use - ASEPT Peritoneal Drainage System

The ASEPT Peritoneal Drainage System is indicated for periodic drainage of recurrent and symptomatic malignant ascites. The catheter is intended for long term access of the peritoneal cavity in order to relieve symptoms such as dyspnea.

#### Indications for Use - ASEPT Pleural Drainage System

The ASEPT Pleural Drainage System is intended for long term, intermittent drainage of symptomatic, recurrent, pleural effusions, including malignant pleural effusions and other pleural effusions that do not respond to treatment of the underlying disease.

#### Contraindications

There are no known contraindications.

## Warnings

- **Do not reuse. Intended for single patient use only.**
- **It is not uncommon to feel some pain or cough when draining fluid. If you feel pain when draining, clamp the drain line to slow or stop the flow of fluid for a few minutes. If you don't feel better after doing this contact your nurse or doctor and withhold further drainage.**
- **Keep the valve on your ASEPT Drainage Catheter clean and the end connector of the drain line sterile.**
- **This product and its packaging have been sterilized with ethylene oxide. Ethylene Oxide is a chemical known to the State of California to cause cancer, birth defects, or reproductive harm.**
- **Do not use scissors or other sharp objects near the ASEPT Drainage Catheter.**

**Note: If the catheter gets punctured or cut, follow the instructions below:**

1. Pinch the catheter closed between your fingers.
2. Push the emergency slide clamp (included in the procedure pack) over the catheter between the patient's body and the cut or leak, then push the catheter completely into the small end of the clamp. The clamp will close the catheter.
3. Contact your doctor or nurse immediately.

## Precautions

- Federal (USA) law restricts this device to sale by or on the order of a physician.
- This product contains dry natural rubber.
- It is recommended to limit drainage to no more than 1000 mL from your chest at any one drainage procedure; 2000 mL from your abdomen during any 24-hour period. The volume of fluid removed should be based on directions from the treating physician.
- Potential complications of access and drainage of the pleural cavity include, but may not be limited to, the following: re-expansion pulmonary edema, pneumothorax, laceration of lung or liver, bleeding, hypotension/circulatory collapse, wound infection, and empyema.
- Potential complications of access and drainage of the abdomen include, but may not be limited to, the following: Laceration of liver or bowel, hypotension/circulatory collapse, electrolyte imbalance, protein depletion, ascites leakage, peritonitis, wound infection, tumor growth in the catheter tunnel, and loculation of the peritoneal space.
- If you see any signs of infection, do not continue with the drainage procedure and contact your doctor or nurse.
- Gauze should remain dry at all times. Patient may shower or bathe once self-adhesive dressing seals gauze from moisture. If gauze appears wet, change dressing immediately.

## Application

Follow these step-by-step directions. Follow physician's direction regarding amount of fluid to be drained.

1. Locate a clean and uncluttered workspace.
2. Wash your hands with soap and water for at least 1 minute.
3. Pick up the bag that contains the ASEPT Drainage Kit. Look at the green vacuum indicator on the top of the bottle. This indicator tells you if there is a vacuum in the bottle. The indicator should be down, not up. If the indicator is up, discard, and use another drainage kit.
4. Peel open the drainage kit pouch all the way and remove the procedure pack. Leave the ASEPT bottle set on top of the pouch.
5. Open the procedure pack bag by peeling the front and back of the pouch apart, starting at the top.
6. Place the procedure pack on your workspace. Remove the tape and unfold the blue wrapping. Leave the enclosed items on the wrapping.
7. Pick up one glove by the wrist and put it on. Pick up the other glove by the wrist and put it on the other hand. Both gloves fit either hand. Be sure the surface of the gloves do not come in contact with anything non-sterile such as clothing or skin. The outside of the gloves needs to remain sterile.
8. Tear open the three alcohol pads but do not remove the pads from the pouches. Place the pads on the blue wrapping. **NOTE:** Do not use alcohol prep pad if dry.

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9. Remove the twist tie that holds together the drain line. Put the bottle back on the open pouch. Do not remove the protective cap on the end of the drain line yet.
10. Remove the adhesive dressing, gauze pads, and foam catheter pad from over your catheter. Check for any redness, tenderness, swelling, or fluid around the catheter.
11. Squeeze the pinch clamp on the drain line completely closed.
12. Hold the catheter valve end in one hand and vigorously clean the valve surface and around the valve with one of the alcohol pads.
13. Remove the protective cap from the ASEPT connector on the drain line and align it in the center of the silicone valve surface.
14. Push the ASEPT connector into the silicone valve and twist to close. The connection must be tight to open the valve and allow the flow of fluid.
15. Release the white vacuum release clamp until it no longer pinches the green tube closed.
16. Release the pinch clamp on the drain line slowly to begin draining fluid. The fluid will flow into the vacuum bottle.

**Note: If there is very little or no fluid flow, check to ensure that the connection between the ASEPT connector and the valve is tight.**

17. Adjust the flow by squeezing the pinch clamp on the drain line partially closed.
18. The flow may slow down when the fluid is almost completely drained. When the flow stops, or the bottle kit is filled with about 2/3 of fluid or more, close the drain line by squeezing the pinch clamp completely closed.

**Note: The drainage typically takes 10-15 minutes.**

19. Clamp the pinch clamp on the drain line fully closed and disconnect the drain line from the valve.
20. Put on the second pair of gloves included in the procedure pack (see **Step 7**).
21. Disinfect the area around the catheter exit site with the remaining two alcohol pads.
22. Remove and discard the framed center panel of the self-adhesive dressing. **NOTE: The framed center panel is on the opposite of the printed side.** Put the dressing back on the blue wrapping.
23. Place the foam catheter pad around the catheter. Curl the catheter around in a circle and place it on top of the foam pad. **Be very careful not to kink the catheter.**
24. Place the gauze pads over the top of the catheter and hold them down.
25. Applying the dressing: Peel the dressing from the printed paper backing, exposing the adhesive surface. View the area where the dressing is to be applied through the film and center the dressing over the wound. Press it down. Do not stretch the dressing during the application. Slowly remove the remaining paper frame by peeling it off on the split section while smoothing down the dressing edges.
26. Once removed, smooth the entire dressing from the center toward the edges and firmly push down the dressing.
27. Remove gloves and discard.

## Advice to Patients

If you still feel shortness of breath, or if the bottle appears to be filled with more than 2/3 of fluid, all the fluid may not have drained. Use a second bottle kit if advised by your physician to do so or contact your doctor or nurse.

## Disposal

Take a pair of scissors and cut off the green rubber tube connected to the drain line as well as the vacuum indicator. Empty the bottle into a toilet, a sink, or in accordance with the facility requirements. Discard the bottle after use.

## STERILITY

This device has been sterilized, and **is for single use only and is not to be reused**. As long as the packaging remains sealed and uncompromised, the contents within each package are sterile. pfm medical will not be responsible for any products that are resterilized, nor accept for exchange or credit any product that has been opened but not used by the patient or purchaser.

## WARRANTY

pfm medical warrants that this medical device is free from defects in both materials and workmanship. The above warranties are in lieu of all other warranties, either expressed or implied, including any warranty of merchantability or fitness for a particular purpose. Suitability for use of the medical device for any surgical procedure shall be determined by the user. pfm medical shall not be liable for incidental or consequential damages of any kind.

For explanation of the symbols on the labeling and packaging, visit [www.pfmmedicalusa.com](http://www.pfmmedicalusa.com)

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