

**pfmmedical**

Quality and Experience

---

**ASEPT<sup>®</sup> Procedure Pack**

---

Instructions for Use

---

[www.pfmmedicalusa.com](http://www.pfmmedicalusa.com)

pfmmedical, the pfm logo and ASEPT are registered trademarks of PFM Medical, Inc.

*Tegaderm<sup>®</sup> is a registered trademark of 3M Company.*

© 2024 PFM Medical, Inc.

**pfmmedical**



PFM Medical, Inc.  
1916 Palomar Oaks Way, Suite 150  
Carlsbad, CA 92008  
Tel.: (760) 758-8749  
Fax: (760) 758-1167  
[customerservice@pfmmedicalusa.com](mailto:customerservice@pfmmedicalusa.com)

LS-00520-01-AC / OCTOBER 2024

# Instructions for Use

## ASEPT Procedure Pack

### Product Description

The ASEPT Procedure Pack is an accessory to the ASEPT Drainage Systems. This product is used to ensure proper sterile wound dressing procedure when draining fluid through an ASEPT catheter. The following instructions are for the use of the ASEPT Procedure Pack alone.

**Before using this product make sure that you have been fully trained on the correct use of the items in this kit. If you have any questions or problems regarding fluid drainage, please contact your doctor or nurse.**

### The ASEPT Procedure Pack contains the following items:

- 2 Pairs of Gloves
- 3 Alcohol Pads
- Foam Catheter Pad
- 6 Gauze Pads
- Tegaderm® Self Adhesive Dressing
- Emergency Slide Clamp

### Indications for Use

The ASEPT Procedure Pack is an accessory to the ASEPT Drainage Systems and includes all dressing change items needed for any ASEPT Drainage Catheter.

### Contraindications

There are no known contraindications.

### Warnings

- Do not reuse. Intended for single patient use only.
- Do not use scissors or other sharp objects near ASEPT Drainage catheter.
- This product and its packaging have been sterilized with ethylene oxide. Ethylene oxide is a chemical known to the State of California to cause cancer, birth defects, or reproductive harm.

### Precautions

- Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.
- If you see any signs of infection, do not continue and contact your doctor or nurse.
- Gauze should remain dry at all times. Patient may shower or bathe once self-adhesive dressing seals gauze from moisture. If gauze appears wet, change dressing immediately.

### Application

1. Locate a clean and uncluttered workspace.
2. Wash your hands with soap and water for at least one minute.
3. Peel open the procedure pack pouch from the top.
4. Place the procedure pack on your workspace. Remove the tape and unfold the blue wrapping. Leave the enclosed items on the wrapping.

5. Pick up one glove by the wrist and put it on. Pick up the other glove by the wrist and put it on the other hand. Both gloves fit either hand. Be sure the surface of the glove does not contact anything non-sterile such as clothing or skin. The outside of the gloves needs to remain sterile.
6. Tear open the three alcohol pads but do not remove the pads from the pouches. Place the pads on the blue wrapping. **NOTE:** Do not use alcohol prep pad if dry.
7. Remove the adhesive dressing, gauze pads and foam catheter pad from over your catheter. Check for any redness, tenderness, swelling or fluid around the catheter.
8. Disinfect the area around the catheter exit site with the alcohol pads.
9. Remove and discard the framed center panel of the self-adhesive dressing. **NOTE: The framed center panel is on the opposite of the printed side.** Put the dressing back on the blue wrapping.
10. Place the foam catheter pad around the catheter. Curl the catheter around in a circle and place it on top of the foam pad. **Be very careful not to kink the catheter.**
11. Place the gauze pads over the top of the catheter and hold them down.
12. Peel the dressing from the printed paper backing, exposing the adhesive surface. View the area where the dressing is to be applied through the film and center the dressing over the wound. Press it down. **Do not stretch the dressing during the application.** Slowly remove the remaining paper frame by peeling it off on the split section while smoothing down the dressing edges.
13. Once removed, smooth the entire dressing from the center toward the edges and firmly push down the dressing.
14. Remove your gloves and discard.

### STERILITY

This device has been sterilized, and **is for single use only and is not to be reused.** As long as the packaging remains sealed and uncompromised the contents within each package are sterile. pfm medical will not be responsible for any products that are re-sterilized, nor accept for exchange or credit any product that has been opened but not used by the patient or purchaser.

### WARRANTY

pfm medical warrants that this medical device is free from defects in both materials and workmanship. The above warranties are in lieu of all other warranties, either expressed or implied, including any warranty of merchantability or fitness for a particular purpose. Suitability for use of the medical device for any surgical procedure shall be determined by the user. pfm medical shall not be liable for incidental or consequential damages of any kind.

**For explanation of the symbols on the labeling and packaging, visit [www.pfmmedicalusa.com](http://www.pfmmedicalusa.com)**